



Gentaur Europe BVBA
Voortstraat 49,
1910 Kampenhout
BELGIUM

Tel 0032 16 58 90 45
info@gentaur.com
<https://maxanim.com/sd-bioline/>

SD BIOLINE Chagas Ab Rapid

One Step antibodies to *Trypanosoma cruzi* Test

Chagas disease, caused by the protozoan parasite *Trypanosoma cruzi*, is a chronic illness affecting about 24million people in Central and South America. In most cases, after an asymptomatic acute phase with parasitemia, parasite growth is controlled by the host immune response. The infection remains quiescent for many years before entering into a chronic phase during which parasites are hardly detectable in the blood of patients. Consequently, detection of specific antibodies in the patient's serum is important for diagnosis of the disease.



General Information

SD BIOLINE Chagas Ab Rapid test is an immunochromatographic screening test for the detection of antibodies to *Trypanosoma cruzi* test in human serum, plasma or whole blood.

- Serological antibody test for a fast and easy diagnosis of the disease
- Specimen : Serum, plasma or whole blood (100 µl)
- Test result : 15 minutes
- Shelf life and storage temperature: 24 months at 1-30 °C
- Performance: Sensitivity - 99.3 %, Specificity - 100 % (vs. ELISA)

Materials Provided


- Test device

Test Procedure

1 Add Specimen

Dispense 100 µl of serum, plasma or whole blood into the sample well "S".

Micropipette

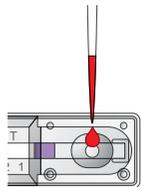

Wait 15 mins.



2 Read Results



Ordering Information

Product	Cat. No.	Type	Specimen	Pack Size
Chagas Ab Rapid	49FK10	Device	Serum/Plasma/Whole Blood	25T/Kit